

QBraille Key Stroke Reference Sheet



ACTION	KEY STROKE	CATEGORY
Move to previous item	Dot 1 + Space bar	Navigation
Move to next item	Dot 4 + Space bar	Navigation
Go to first item	Dot 1 + Dot 2 + Dot 3 + Space bar	Navigation
Go to last item	Dot 4 + Dot 5 + Dot 6 + Space bar	Navigation
Item chooser	Dot 2 + Dot 4 + Space Bar	Navigation
Go to Status bar	Dot 2 + Dot 3 + Dot 4 + Space bar	Navigation
Go to Notification Center	Dot 4 + Dot 6 + Space bar	Navigation
Go to Control Center	Dot 2 + Dot 5 + Space bar	Navigation
Escape current context	Dot 1 + Dot 2 + Space bar	Navigation
Move to Left Split View App (iPad only)	Dot 3 + Dot 5 + Space bar	Navigation
Move to Right Split View App (iPad only)	Dot 2 + Dot 6 + Space bar	Navigation
Move to previous container (iPad only)	Dot 1 + Dot 7 + Space bar	Navigation
Move to next container (iPad only)	Dot 4 + Dot 7 + Space bar	Navigation
Scroll left one page	Dot 2 + Dot 4 + Dot 6 + Space bar	Scrolling
Scroll right one page	Dot 1 + Dot 3 + Dot 5 + Space bar	Scrolling
Scroll up one page	Dot 3 + Dot 4 + Dot 5 + Dot 6 + Space bar	Scrolling
Scroll down one page	Dot 1 + Dot 4 + Dot 5 + Dot 6 + Space bar	Scrolling
Speak page number or rows being displayed	Dot 3 + Dot 4 + Space bar	Scrolling
Move to previous item using rotor setting	Dot 3 + Space bar	Rotor
Move to next item using rotor setting	Dot 6 + Space bar	Rotor

ACTION	KEY STROKE	CATEGORY
Select previous rotor setting	Dot 2 + Dot 3 + Space bar	
Select next rotor setting	Dot 5 + Dot 6 + Space bar	
Perform simple tap	Dot 3 + Dot 6 + Space bar	Interaction
Activates the Home button	Dot 1 + Dot 2 + Dot 5 + Space bar	
Activates the Volume Up button	Dot 3 + Dot 4 + Dot 5 + Space bar	
Activates the Volume Down button	Dot 1 + Dot 2 + Dot 6 + Space bar	
Show/Hide the keyboard	Dot 1 + Dot 4 + Dot 6 + Space bar	
3D Touch on selected item	Dot 3 + Dot 5 + Dot 6 + Space bar	
Read page starting at selected item	Dot 1 + Dot 2 + Dot 3 + Dot 5 + Space bar	Reading
Read page starting at the top	Dot 2 + Dot 4 + Dot 5 + Dot 6 + Space bar	
Select all	Dot 2 + Dot 3 + Dot 5 + Dot 6 + Space bar	Editing
Select left	Dot 2 + Dot 3 + Dot 5 + Space bar	
Select right	Dot 2 + Dot 5 + Dot 6 + Space bar	
Tab	Dot 2 + Dot 3 + Dot 4 + Dot 5 + Space bar	
Shift Tab	Dot 1 + Dot 2 + Dot 5 + Dot 6 + Space bar	
Cut	Dot 1 + Dot 3 + Dot 4 + Dot 6 + Space bar	
Copy	Dot 1 + Dot 4 + Space bar	
Paste	Dot 1 + Dot 2 + Dot 3 + Dot 6 + Space bar	
Activates the Delete key	Space bar or Dot 1 + Dot 4 + Dot 5 + Space bar	
Activates the Return key	Space bar or Dot 1 + Dot 5 + Space bar	
Undo Typing	Dot 1 + Dot 3 + Dot 5 + Dot 6 + Space bar	
Redo Typing	Dot 2 + Dot 3 + Dot 4 + Dot 6 + Space bar	
Text search	Dot 1 + Dot 2 + Dot 4 + Space bar	

ACTION	KEY STROKE	CATEGORY
Output text style	Dot 2 + Dot 3 + Dot 4 + Dot 5 + Dot 6 + Space bar	Rotor
Start dictation when in text field	Dot 1 + Dot 5 + Dot 6 + Space bar	Rotor
Toggle Screen Curtain on and off	Dot 1 + Dot 2 + Dot 3 + Dot 4 + Dot 5 + Dot 6 + Space bar	Control
Pause or continue speech	Dot 1 + Dot 2 + Dot 3 + Dot 4 + Space bar	Control
Toggle speech on or off	Dot 1 + Dot 3 + Dot 4 + Space bar	Control
App Switcher	Dot 1 + Dot 2 + Dot 5 + Space bar (twice)	Control
Help	Dot 1 + Dot 3 + Space bar	Control
Change an item's label	Dot 1 + Dot 2 + Dot 3 + Dot 4 + Dot 6 + Space bar	Control
Toggle QuickNav	Dot 1 + Dot 2 + Dot 3 + Dot 4 + Dot 5 + Space bar	Control
Play/Pause Music	Dot 1 + Dot 5 + Dot 6 + Space bar	Control
Pan braille to the left	Dot 2 + Space bar	Braille
Pan braille to the right	Dot 5 + Space bar	Braille
Toggle announcement history	Dot 1 + Dot 3 + Dot 4 + Dot 5 + Space bar	Braille
Translate	Dot 4 + Dot 5 + Space bar	Braille
Next output mode	Dot 1 + Dot 2 + Dot 4 + Dot 5 + Space bar	Braille
Next input mode	Dot 2 + Dot 3 + Dot 6 + Space bar	Braille
Switch between contracted and uncontracted braille	Dot 1 + Dot 2 + Dot 4 + Dot 5 + Space bar	Braille
Previous container	Dot 1 + Dot 7 + Space bar	8-dot
Next container	Dot 4 + Dot 7 + Space bar	8-dot
Delete	Dot 7 + Space bar	8-dot
Return	Dot 8 + Space bar	8-dot
Toggle mute	Dot 1 + Dot 3 + Dot 4 + Dot 7 + Space bar	8-dot

ACTION	KEY STROKE	CATEGORY
Start help	Dot 1 + Dot 3 + Dot 7 + Space bar	8-dot
Scroll up page	Dot 3 + Dot 4 + Dot 5 + Dot 6 + Dot 8 + Space bar	8-dot
Scroll down page	Dot 1 + Dot 4 + Dot 5 + Dot 6 + Dot 8 + Space bar	8-dot
Simulated long press	Dot 3 + Dot 6 + Dot 7 + Dot 8 + Space bar	8-dot
Single letter quick nav	Dot 1 + Dot 2 + Dot 3 + Dot 4 + Dot 5 + Dot 7 + Space bar	8-dot



HIMS, inc.

Website: <https://www.hims-inc.com>

Phone: 512-837-2000

Email: sales@hims-inc.com